

Cementation & Bonding Protocol

FirstFit Guided Bridge System

Cementation Protocol

- ✓ Isolate the prepared teeth
- ✓ Etch the teeth for 15 to 20 seconds
- ✓ Thoroughly rinse the etched surfaces
- ✓ Apply a layer of chlorhexidine
- ✓ Suction humidity
- ✓ Apply a MDP containing bonding agent such as Scotchbond Universal
- ✓ Cement with luting cement of choice such as Rely X

Bonding Protocol

- ✓ Bond with a warm posterior Class 2 Composite of choice
- ✓ The connectors or the surfaces of the restoration that is being bonded should be treated with a primer containing MDP

FirstFit Guided Crown System

- ✓ Follow general protocols and techniques to cement a single unit crown

FIRSTFIT™

guided prosthetics delivery system

1.844.201.6329 • www.firstfit.com

Bonding Protocol

FirstFit Guided Veneer System

- ✓ Add one coat bonding agent to veneers
- ✓ Isolate the teeth
- ✓ Clean the teeth with pumice
- ✓ Etch teeth for 20 seconds
- ✓ Apply bonding agent
- ✓ Load veneers with luting cement
- ✓ Insert the tray
- ✓ Use micro brush to remove excess cement at margins
- ✓ Check to ensure all margins are seated
- ✓ Tack cure for two seconds per tooth
- ✓ Do additional clean up where necessary
- ✓ Cure each veneer for ten seconds
- ✓ Remove tray
- ✓ Use Cerisaw to clean inter proximal surfaces
- ✓ Final cure each veneer on facial and lingual for 20 seconds
- ✓ Check and adjust occlusion

FIRST FIT™

guided prosthetics delivery system

1.844.201.6329 • www.firstfit.com