

# FIRSTFIT™

Powered by  HENRY SCHEIN®



## FirstFit Guided Veneer Delivery System

### *Pro* **Tips**

---

- ✓ Soak the guides in warm water for 30-40 seconds before use to make them easier to work with
- ✓ Position guides in number order on your tray to allow easy transitions from one guide to the next
- ✓ Test each guide with handpiece before inserting the bur to ensure disc slides easily into each guide
- ✓ Maximize ease of use by using a retractor to isolate teeth and reference the midline to line up guides
- ✓ Direct the patient to turn their head to ensure you have a clear view of the handpiece's direct insertion into the guides
- ✓ When preparation is complete, go back and forth three to four more times to ensure the preparation is fully complete. Also note that if fully prepped, bur should no longer be moving— if bur is still moving, please run guides through once more
- ✓ Freehand with the bur any shiny facial surface ensuring all enamel has been minimally prepped (could be less than .3mm). Also make sure incisal edges are smooth prior to using delivery tray pre-loaded with veneers
- ✓ If the veneers separate from the delivery tray, manually place the loose units onto the already prepped preview model, mix light cure, and place the uncured light cure into the tray and slide it down over the preview model. Light cure for the appropriate amount of time required by the specific composite product used